

STEP 2

Fit the different impression trays on your teeth, until you decide which size to use (Small, Medium or Large).

The Tray should overlap all your teeth and gums without hitting the gum at any point or on the other hand being so wide.

STEP 3

Get the impression material out of one of the containers with lable 1 & one of the containers with lable 2 and mix them quickly by kneading and folding for 40 seconds or until it becomes a solid one homogeneous color.

0



STEP 4

Roll the putty and mak it as tall as your finger and then load the tray evenly and fully with it



Try to swallo possible.

STEP 6

Insert the tray in your mouth. Mal fully inside your mouth before touching any ter

STEP 7

against your 1 straight line v rocking or wi



y press arou make a per

STEP 9

INTERNET






put it along with impression back in the box. Put the box in the return shipping bag and send it over.

Home impression making is a very simple process. Perfect impress Veneers, that's why sometimes we would send you another impress the impression again to get more details or a more accurate im h = Perfect

Few Examples for Good Impressions:

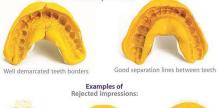



In Most of the cases it and teeth to make the minutes to take those , pl



Return Card

e, Don't forg Full Name :

Full Address :

Phone :

Email : By Signing this card and sending y and Conditions available online. ee to all S u aqr

Signature : Date :